

# VacuValve™

## HVE Lite Kit

SKU: VVL3



### Kit Includes:

- (3) Compact HVE Valve Assemblies
- (1) Swivel Quick Disconnect
- (1) Cleaning Brush




**Ascentcare**  
Dental Products

[AscentcareDental.com](http://AscentcareDental.com)

# VacuValve™ HVE Lite

## USER GUIDE

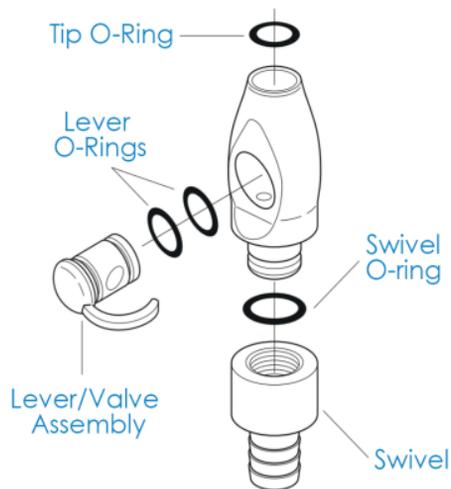

### MAINTENANCE:

To ensure smooth lever action and quick disconnect swivel, lubricate O-rings regularly\*

Replace any O-ring that becomes worn, nicked, or cut.

*\*Always use silicone-based lubricants, such as Ascentcare's O-Ring Lubricant, available in the HVE Valve Maintenance Kit (SKU: HVEMK)*

### CLEANING:

Use a stiff brush and mild detergent to clean internal surfaces.

### DISINFECTING:

Run the VacuLUX HVE Adapter with a non-corrosive, pH-neutral enzymatic cleaning solution and perform a surface disinfection with a disinfectant wipe.

### STERILIZING:

Products should be cleaned as soon as possible after use to prevent soils from hardening. Remove all visible debris prior to sterilizing. Process product through steam autoclave following the autoclave manufacturer's instructions. Do not exceed a temperature of 132°C/270°F. 6 minutes holding time, minimum. Avoid contact of sterilizer wall and avoid close proximity to heating elements. Store only in FDA approved sterilization pouch.



For customer support:  
call: (616) 600-4505

email: [support@ascentcaredental.com](mailto:support@ascentcaredental.com)

Manufactured by Ascentcare Dental Products, Inc.