



# VacuVUE

HVE DENTAL MIRROR

## Instructions For Use


### USING A STANDARD HVE VALVE

Simply & securely connect the VacuVUE to your existing HVE valve like a standard suction tip.

### USING YOUR OPTIONAL FEATHER FLEX™ KIT

For a lightweight feel, improved maneuverability & reduced fatigue, use the Feather Flex™ Kit.

1. Securely insert the curved stainless-steel tube into your HVE valve like a suction tip.
2. Rest your valve in its docking stand.
3. Connect your VacuVUE™ mirror to the aluminum coupler which allows for 360° rotation and maintains an air-tight seal.

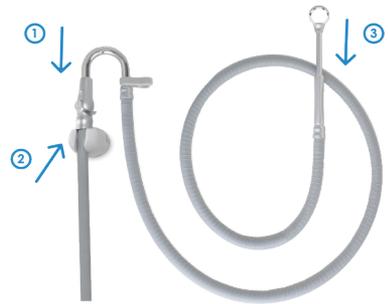

The Feather Flex™ Kit  
SKU: FFK5

### INTENDED USE:

The VacuVUE™ HVE mirror is intended to provide oral high-volume evacuation, visualization and retraction during dental procedures. The device is intended to be used by trained dental professionals only.

### INSTRUCTIONS FOR USE:

Before patient use, the VacuVUE™ mirror must be cleaned & sterilized. Follow sterilization instructions as directed below.

1. Ensure that the mirror is properly and securely installed by following installation instructions.
2. Ensure that your HVE suction system is turned on.

3. On your HVE valve, switch the lever from off to on.

4. Once on, suction should flow through the VacuVUE™ mirror evacuation ports and the mirror is ready to use.

5. Operate the on/off lever on your HVE valve as needed throughout your procedures.

To install:

To use:

# VacuVUE AUTOCLAVE INSTRUCTIONS

Products should be cleaned & sterilized between each patient. Clean as soon as possible to prevent soils from hardening.

1. Remove all visible debris. It is recommended to do so using a stiff brush and mild detergent to clean internal surfaces. Ultrasonic cleaners or automatic instrument washers may also be used to further remove debris after brushing. Follow the manufacturer's instructions. Rinse mirrors before autoclaving.

**\*Do not autoclave the Feather Flex™ Kit.** Refer to the instructions for use included with your Feather Flex™ Kit for separate sterilization instructions.

## MIRROR FACE REPLACEMENT

1. To remove the old mirror, using gloved thumbs, carefully apply even pressure to the mirror face until the back-plate pops free.



① Use both thumbs for even pressure

3. Process mirrors through steam autoclave\* at a temperature between 121°C/250°F and 135°C/275°F for a minimum of 15 minutes. Do not exceed 135°C/275°F.

4. Avoid contact of sterilizer wall and avoid close proximity to heating elements.

5. Store only in FDA approved sterilization pouch.

2. Remove the O-ring with a tool such as a small scaler.

3. Remove and discard the old mirror.



② Replace O-Ring for optimal seal

4. Replace with a new mirror, **beveled side facing down**.

5. Insert a new O-ring into the track firmly and lubricate.



⑤ O-ring



④ mirror face, **bevel side down**



6. On a hard surface, line up the back plate with the mirror, pegs upward.

7. While applying even pressure with two thumbs, push the mirror head down over the backplate until it pops securely into place.



⑦ Use both thumbs for even pressure

**ATTENTION: Maintenance required.** To ensure ease of back plate removal, be sure to regularly lubricate the O-Ring seal by removing the back plate and rubbing O-Ring lubricant around the rubber seal. **Follow this procedure every 15-20 autoclave cycles.**

[www.AscentcareDental.com/hve-mirror](http://www.AscentcareDental.com/hve-mirror)

Visit our website to learn more about your  
VacuVUE™ HVE Mirror



335 North Griffin Street  
Grand Haven, MI 49417

**(616) 600-4505**