



Radical Reflector™ Lighted Mouth Mirror

1. When sterilized and ready to use, couple the magnetic end of the Radical Reflector to the fiber optic light guide via the magnetic coupler.
2. Insert the other end of the fiber optic light guide into the HyperLUX™ Light Engine and turn on according to the HyperLUX light engine instructions.
3. When finished, remove from the fiber optic light guide and sterilize according to the instructions in this pamphlet.

Ascentcare™
Dental Products

www.ascentcaredental.com

Sterilization Instructions:

Remove from packaging and sterilize all products according to these instructions before use.

Sterilization should be performed using chemical methods or via autoclave. Chemical sterilization is the recommended method of sterilization for added longevity of your products.

Autoclave Sterilization:

Please note: Insufficient water to the autoclave could cause parts to melt.

1. Products should be cleaned as soon as possible after use to prevent soils from hardening. Remove all visible debris prior to sterilizing.
2. Process product through steam autoclave following the autoclave manufacturer's instructions. Do not exceed a temperature of 136°C/275°F.
3. Store only in FDA approved sterilization pouch.

Chemical Methods:

1. As soon as possible after use, soak and rinse your Ascentcare instruments in warm water to remove any blood or tissues.
2. Select a liquid chemical agent registered with the EPA as a "sterilant."
3. Follow the sterilization instructions of your sterilant liquid. Most chemical sterilization solutions render instruments sterile only after 10-hour immersion.
4. Rinse, dry and store in a FDA approved sterile container if not used immediately.

The Radical Reflector is latex free.

As with all products containing magnets, please take care to avoid inadvertently deactivating such devices as pacemakers, ICD's and other implanted medical devices.

Ascentcare's products are protected by US Patents. For patent information, visit:
www.ascentcaredental.com/patents

(800) 500-0533
service@ascentcaredental.com